



SOS

Speciality Oxygen Service



ProTrainings.uk

www.protrainings.uk
www.first-aid-online.co.uk



Nitronox

User Instructions

Check the gauge to see how much Nitronox is in the cylinder. Contact us for a refill when the needle is in the red section.



Insert a compatible demand valve into the quick connect port on the right side of the valve (push until you feel a 'click').



Turn the on/off switch on the left side of the valve anti-clockwise to the 'ON' position.



Nitronox will be automatically administered by the demand valve when it detects a breath.



After use, turn the on/off switch clockwise to the 'OFF' position.



Press the blue dot on the demand valve to expel any excess Nitronox that is left in the hose. Disconnect the demand valve by pushing the outer ring of the quick connect port.

