



Medical Oxygen

User Instructions


Check the gauge to see how much oxygen is in the cylinder. Contact us for a refill when the needle is in the red section.



Attach the mask to the outlet above the gauge.



Alternatively, insert a compatible device into the quick connect port on the right side of the valve (push until you feel a 'click').

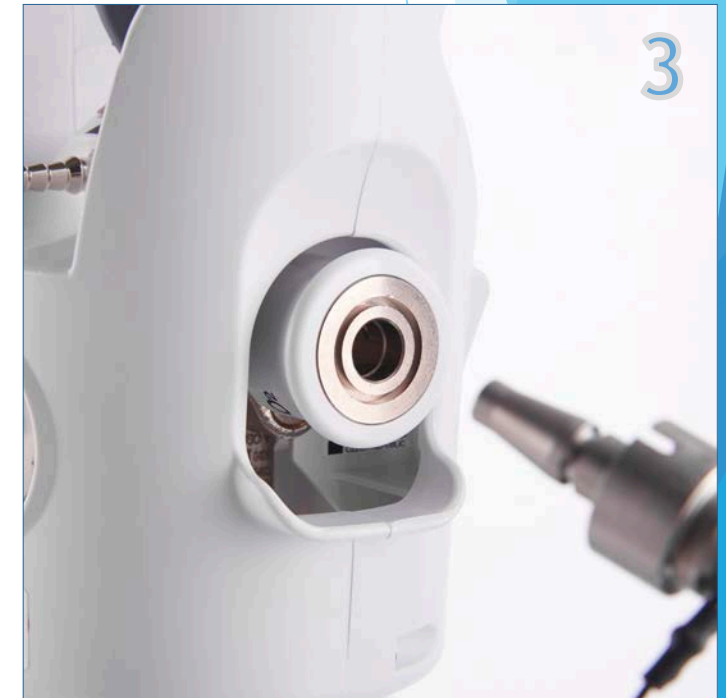

Turn the flow rate selector anti-clockwise to match the black arrow with the required flow rate.



To stop the flow of oxygen, turn the flow rate selector clockwise to the off position. Make sure it is turned all the way.



If a compatible device was used, disconnect it by pushing the outer ring of the quick connect port.

